

VistaVision Software Updates

Buy a VUP together the VistaVision! You could use the free software update forever!

What is a VUP and why is it useful?

The VUP is a programmed USB flash drive you can use:

- **to update easier and faster your VistaVision software**
- **as a "rescue disk", to restore the VistaVision in case of problems**

The VUP is the low cost system you must buy, if you want to maintain updated your VistaVision: insert this request in your purchase order.



How does it work?

VistaVision upgrade is very easy if you use a VUP: insert the VUP and power on the instrument, that's all.

All you need is a VUP provided by DMD Med Tech, so the distributor or the final user could update VistaVision directly where it is installed, without an expert help... follow these instructions:

- 1) **with your VistaVision turned off, insert the VUP into the USB port**
- 2) **turn on the VistaVision by pressing a remote control key, then wait for update ended (six-ten minutes)**
- 3) **when update was ended the VistaVision will be automatically turned off (please, remove the VUP)**



The software upgrade restores all default preferences: we suggest to take note of all customization choices (patient-screen distance, mirror, etc...) beforehand. Once the upgrading procedure is completed, repeat the "Guided customization" to set up the VistaVision software.

What do I have to do when the VUP software is old?

Every time a new software is released, DMD Med Tech make it available, free of charge, for VUP owners through internet download. An account is required: please get your registration on the producer's website <http://www.dmdmedtech.it> (Support -> Downloads).

If you have a VUP with an obsolete software, you must update a VUP before update your VistaVision:

- 1) **make a login and download the last available software from 'Support-Downloads' area on <http://www.dmdmedtech.it>**
- 2) **copy the downloaded file to your VUP. Attention: you must replace the old file!**

How to know the current software version of VistaVision? To be sure about current version installed on your VistaVision, please move the selection on INFO button (SETUP window): the software version will be displayed on the bottom information box

VISTAVISION® is a registered trademark of DMD Med Tech s.r.l.



VistaVision is engineered and produced by DMD Med Tech srl

Via Monviso, 14 – 10090 Villarbasse Torino Italy; Tel ++39 0119528282; Fax ++39 0119528402

www.dmdmedtech.it, medtech@dmd.it

DMD
MED TECH